

BC-BF

HEMATOLOGY CONTROL

Reference Values provided by DYMIND
CONTROL
 2025-10-13

 2026-01-10

Applicable Instruments	Parameter	Unit	LOT BFI1125A-1	LOT BFI1125A-2	LOT BFI1125A-3
DYMIND DH-800 series DH-800 CRP series DH-800 CS series (Technical File Version A1.3 or higher)	WBC-BF	10 ⁹ /L	0.125 ±0.050	0.266 ±0.150	0.912 ±0.200
	RBC-BF	10 ¹² /L	0.029 ±0.020	0.080 ±0.060	0.480 ±0.100

【NOTE】

1. The controls should be stored in refrigerator (2℃~8℃). After opening, it will keep stable for 30 days when it is stored airtight at 2℃~8℃.
2. Before mixing and running the control after take it out from the refrigerator, please keep it at least 15 minutes until reaching room temperature(15℃~30℃).
3. Controls must be well mixed before using. Please mix gently, to avoid cells rupture and/or generating bubbles.
4. After using, put the controls back into the refrigerator to prevent contamination and evaporation.



BFI1125A-1



BFI1125A-2



BFI1125A-3